



LTV<sup>®</sup> 1200 Ventilator  
Emergency set-up card

**Step 1: Choose a power source.**


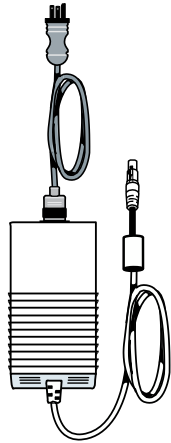


AC adapter  
(Use AC power when available).

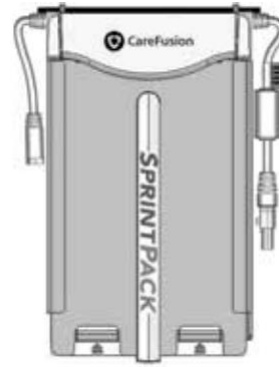


SprintPack Lithium-Ion  
Power System

**Step 2: Connect the LTV® 1200 ventilator to the power source.**

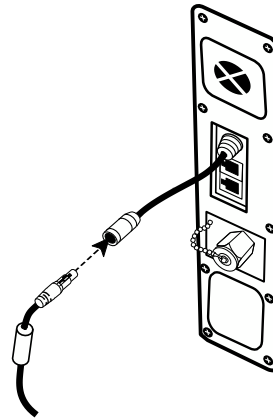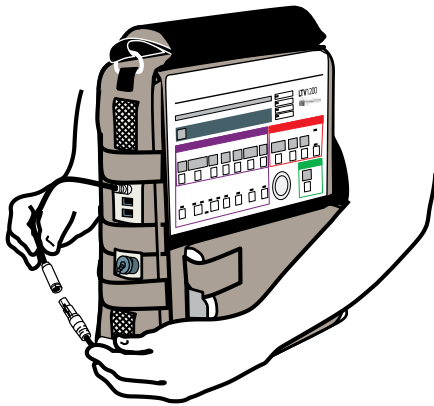

**Step 3: Choose a circuit.**

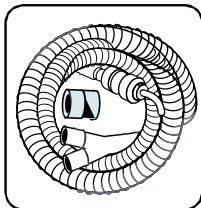

- Open bag.
- Remove patient circuit and 22 mm adapter from bag.

Humidifier circuit (optional)




22 mm patient circuit, **adult**

>20 kg

>44 lb

22 mm adapter

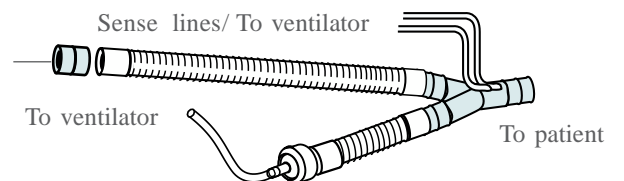


15 mm patient circuit, **infant/pediatric**

5 kg to ≤ 20 kg


11 lb ≤ 44 lb

22 mm adapter

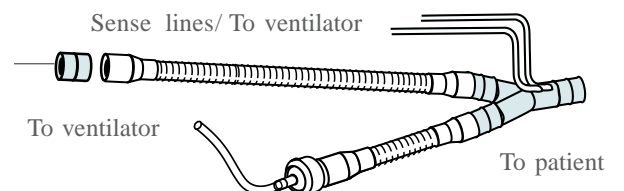


For 24-hour assistance, call 800.754.1914


**Step 4: Connect the circuit to the LTV 1200 ventilator.**

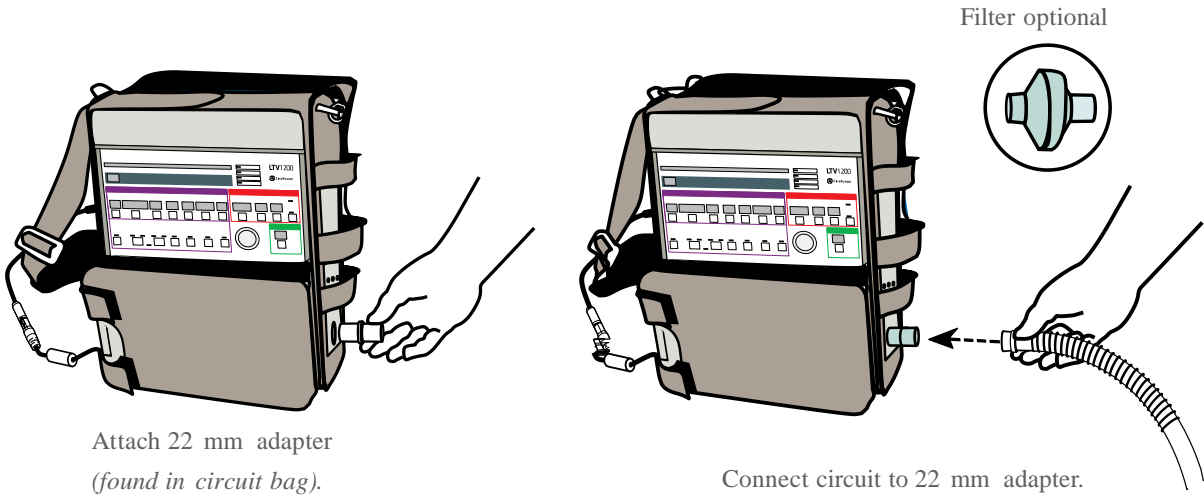


**Step 5: Connect the sense lines to the LTV 1200 ventilator.**

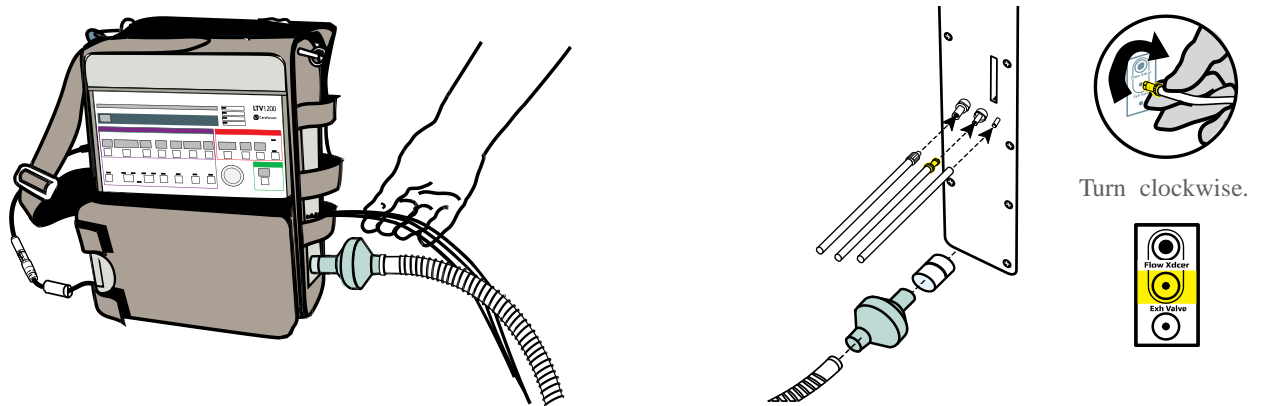

**Step 6: Turn on the LTV 1200 ventilator.**

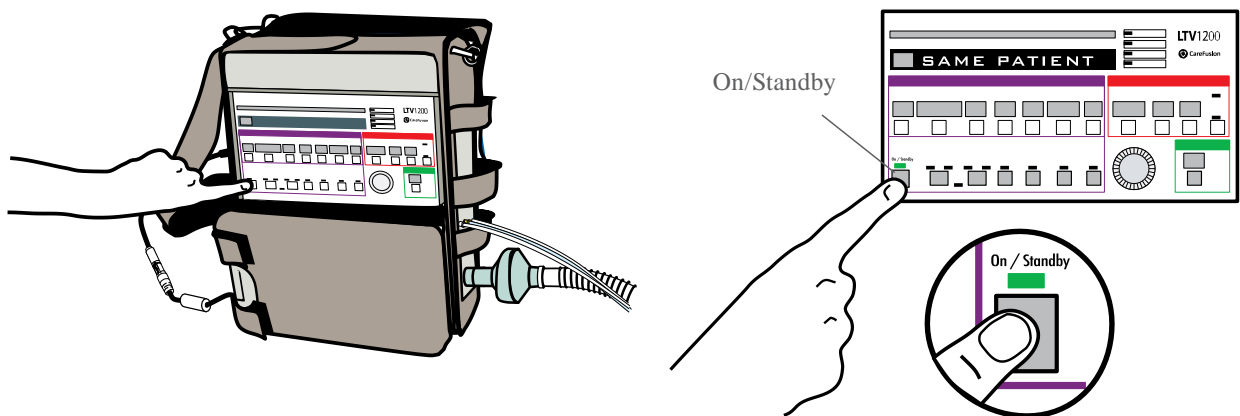


**Step 7: Patient setting displays.**

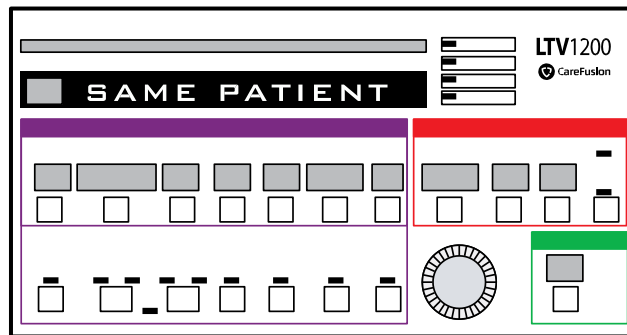


Once the power is on, see **Same Patient** display.

**Step 8: Change display to the New Patient setting.**

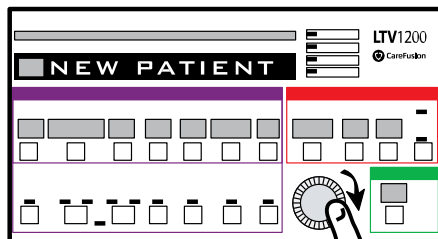


Rotate **Set Value** knob clockwise until **New Patient** is displayed.

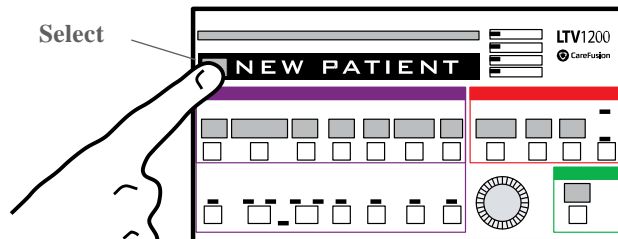


Press **Select** when **New Patient** is displayed.

**Step 9: Set patient size.**

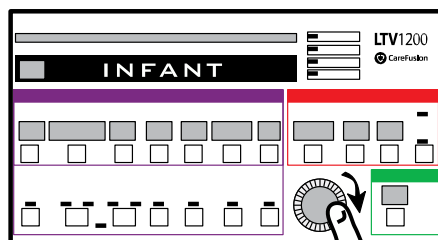


Rotate **Set Value** knob to correct patient size (*infant, pediatric or adult*).

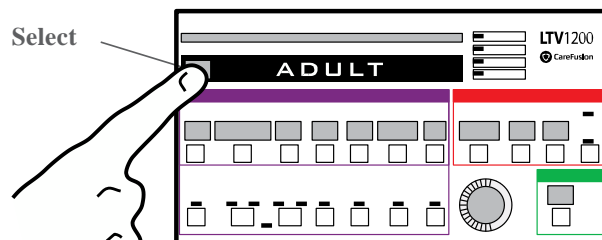

Press **Select** when proper patient size is displayed.


**Note:** If you make a mistake, turn the LTV 1200 ventilator off by holding the On/Standby button for three seconds. Then return to Step 6.

**Step 10: Connect circuit to ET tube or mask.**

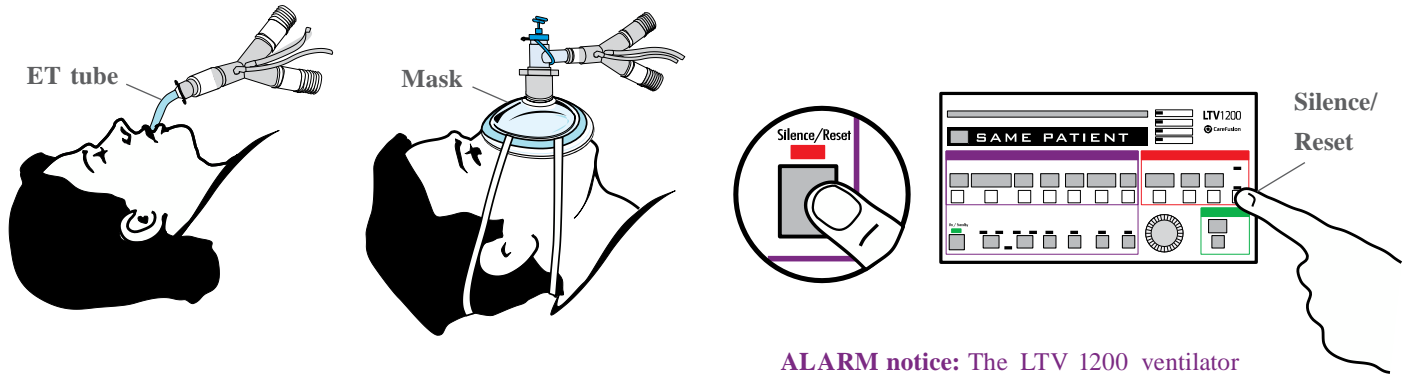


**ALARM notice:** The LTV 1200 ventilator will alarm until it is connected to the patient. To turn it off, you may press the **Silence/Reset** button.

**Step 11: Optional: Use a mask.**

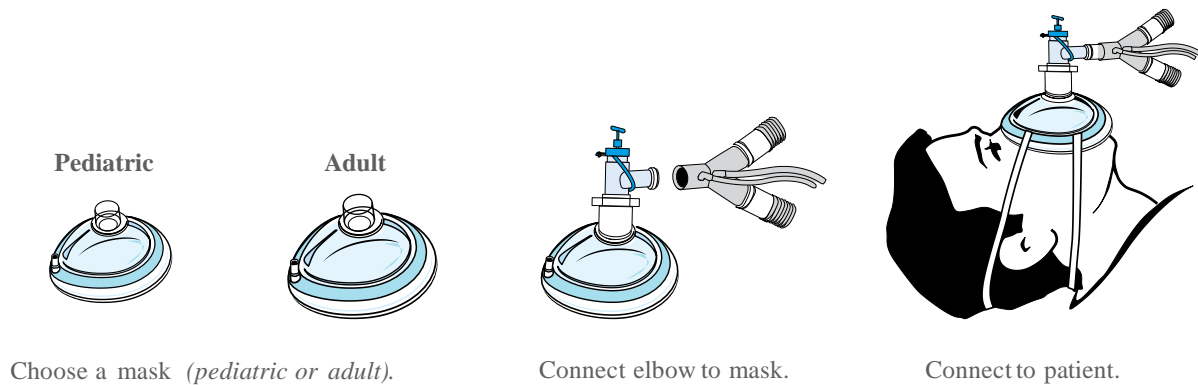

**Step 12: Optional: Use an HME and/or flextube as shown for patient comfort.**

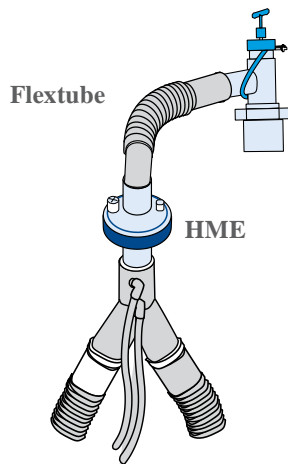

This emergency set-up card is intended to provide emergency instructions for initial LTV 1200 ventilator set-up. The initial settings may need to be adjusted based on the patient's clinical condition. For complete information regarding set-up and operation, refer to the LTV 1200 ventilator operator's manual.



[carefusion.com](http://carefusion.com)

